MASTAZYME EBV (EBNA-1)

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11. Literatur

1. Chow, K.C. et al. Serum responses to the combination of Epstein-Barr virus antigens from both latent and acute phases in nasopharyngeal carcinoma: complementary test of EBNA-1 with EA-D. *Cancer Epidemiol. Biomarkers Prev.*, **6**: 363 (1997).
2. Debyser, Z. et al. Comparative evaluation of three ELISA techniques and an indirect immunofluorescence assay for the serological diagnosis of Epstein-Barr virus infection. *Clin. Diagn. Virol.*, **8**: 71 (1997).
3. Dobec, M. et al. Evaluation of a new Epstein-Barr virus Combi Test for rapid serologic diagnosis of infectious mononucleosis. *Zentralbl. Bakteriol.*, **284**: 565 (1996).
4. Dopatka, H.D. et al. Compact Epstein-Barr virus diagnosis based on a defined antigen mix and specific IgA. *Res. Virol.*, **147**: 53 (1996).
5. Korycakova, L. Epstein-Barr virus antibodies: review of results from repeated examinations in 208 hemodialyzed patients. *Vnitr. Lek.*, **41**: 377 (1995).
6. Nebel-Schickel, H. et al. Anti-EBNA-1 (carboxy-half) IgG antibodies as a seroepidemiological marker for Epstein-Barr virus infection. *Beitr. Infusionsther. Transfusionsmed.*, **32**: 134 (1994).
7. Obel, N. et al. Serological and clinical findings in patients with serological evidence of reactivated Epstein-Barr virus infection. *APMIS*, **104**: 424 (1996).
8. Rivero, N. et al. Evaluation of the detection of IgM by EIA against the p18 protein of the IgG capsid against EBNA in the diagnosis of acute Epstein-Barr infection. *Enferm. Infec. Micro-biol. Clin.*, **16**: 45 (1998).
9. Ternyak, G. et al. The serological signs of the Epstein-Barr virus (EBV) activity in the elderly. *Acta Microbiol. Immunol. Hung.*, **44**: 133 (1997).
10. Weissbrich, B. The use of semi-automated EBV IgG avidity determination for the diagnosis of infectious mononucleosis. *J. Med. Virol.*, **54**: 145 (1998).

Contents	Page
1. Intended Use	10
2. Introduction	10
3. Principle of the Test	10
4. Kit Contents	11
5. Materials Required but not Provided	11
6. Warnings and Precautions	12
7. Storage and Stability	12
8. Specimen Collection and Handling	12
9. Assay Procedure	13
10. Results and Interpretation	14
11. Assay Performance	14
12. References	15

1. Intended Use

MASTAZYME EBV (EBNA-1) Antibody ELISA has been designed for the detection and the quantification of specific IgG / IgM / IgA antibodies respectively against nuclear antigen of Epstein-Barr virus in serum and plasma. Further applications in other body fluids are possible and can be provided on request.

This assay is intended for *in-vitro* diagnostic use only.

All laboratory test results should be interpreted in conjunction with other clinical data. The clinical judgement and further tests have to be taken into account additionally.

2. Introduction

In 1964, Epstein, Barr and Achong characterised by electron microscopy as the causing agent a hitherto unknown virus which belongs to the family of herpes viruses. The Epstein-Barr virus is made responsible for a variety of diseases like infectious mononucleosis, Burkitt lymphoma, as well as nasopharyngeal carcinoma. In addition, a role of the virus is discussed in connection with Hodgkin's disease. Especially with teenagers there appears a glandular fever syndrome which is called „kissing disease“.

Diseases which are caused by the Epstein-Barr virus are found mainly in persons with reduced immunity. For example, the virus is associated with a lympho-proliferative disease which occurs after transplantation. The immune system of such patients is usually impaired by drug therapy. Also in immune-deficient AIDS patients there appears frequently a state where cells at the rim of the tongue are infected (oral hairy leukoplakia).

Infected people keep the Epstein-Barr virus forever in their body. They are, however, mostly not ill. In the developing countries practically all the people are infected, in the western world the incidence is between 80 % and 90 %. The transmittance occurs already during childhood perhaps by transfer from the mother mainly via the saliva.

During the active phase of the viral cycle the Epstein-Barr virus produces about 100 different anti-gens in the inactive phase around 10. The latter comprises among others the EBV nuclear antigen EBNA-1, which is closely correlated with a past infection and an immunity. The early antigen (EA) as well as the virus capsid antigen (VCA) from the active phase are also used as diagnostic markers.

In a fresh infection, IgM antibodies against VCA and EA are determined by immunofluorescence or ELISA. Later VCA IgG and afterwards EBNA-1 IgG antibodies appear. The simultaneous activation of VCA IgM and EBNA-1 IgG indicates correspondingly a reactivation of a latent EBV infection.

3. Principle of the Test

The principle of the test reaction can be described in four stages.

3.1 Serum incubation

Specific antibodies bind to the antigens on the solid phase to form a stable immune complex. After a 60 minutes incubation at room temperature the wells are washed with prediluted wash buffer to remove all non-reactive serum components.

3.2 Conjugate incubation

The anti-human-IgG /-IgM /-IgA horseradish peroxidase conjugate is added to all wells. The conjugate binds to IgG / IgM / IgA antibodies on the solid phase antigen to form a stable sandwich. After a 30 minutes incubation at room temperature the excess conjugate is removed by washing all wells with washing buffer.

3.3 Substrate reaction and stopping

The TMB substrate is dispensed into each well and the peroxidase enzyme/substrate reaction forms a stable blue chromogen. The reaction and subsequently the colour development is stopped after 20 min incubation at room temperature by adding 0.5 N H₂SO₄ to the wells. The change in pH also causes the chromogen to change colour from blue to yellow.

3.4 Reading and interpretation

The intensity of the colour is read in a microtiter plate reader at 450 nm (recommended reference wavelength for bichromatic measurement: 600 - 690 nm). The intensity of the colour (OD) is directly proportional to the concentration of the specific antibody in the patient sample.

4. Kit Contents

The kits contains sufficient reagents for 12 x 8 = 96 determinations. The strips and solutions have to be stored at 4 - 8 °C. The expiry date is mentioned on the labels.

12 strips	Microtiter strips	each with 8 break-apart wells coated with the nuclear antigen (EBNA-1) of Epstein-Barr virus
1 x	Frame holder	
4 x 2 mL	Calibrators 1 - 4	human serum containing antibodies against EBV (EBNA-1) (concentrations listed below) diluted in PBS and stabilised with 0.01 % methylisothiazolone and 0.01 % bromonitrodioxane as preservatives, ready to use.

		IgG	IgM	IgA
Cal. 1 (negative)	Concentration (U/mL)	1	1	1
Cal. 2 (Cut-off)		10	10	10
Cal. 3 (weak positive)		50	30	50
Cal. 4 (positive)		150	125	150

1 x 60 mL	Serum diluent	PBS/BSA buffer solution, contains < 0.1 % sodium azide as preservative, ready to use
1 x 12 mL	Enzyme conjugate solution	HRP-labelled goat anti-human-IgG /-IgM /-IgA, ready to use
1 x 12 mL	TMB substrate	3,3',5,5' Tetramethylbenzidine, ready to use
1 x 12 mL	Stopping solution	0.5 N sulfuric acid, ready to use
1 x 60 mL	Washing buffer	PBS/Tween buffer solution 10x concentrated to be diluted 1:10 prior to use; the concentrate should be warmed up to 37 °C for 15 min to avoid any crystals
2 x	Plate sealers	to cover microtiter strips during incubation
1 x	Plastic bag	re-sealable for dry storage of non-used strips

5. Materials Required but not Provided

- 5 µL-, 100 µL- and 500-µL micro- and multichannel pipets
- Microtiter plate reader with a 450 nm filter (reference filter 600 – 690 nm)
- Microtiter Plate Washer (in case of manual washing: wash bottle)
- Reagent tubes for the serum dilution
- Measuring cylinder
- Distilled water or water of higher quality

6. Warning and Precautions

- For *in-vitro* diagnostic use only! Do not ingest or swallow! Laboratory safety precautions should be followed. Do not eat, drink or smoke in the laboratory.
- All sera and plasma or buffers based upon have been tested to HBsAg, HIV and HCV respectively with generally accepted methods and were found negative. Nevertheless, precautions like the use of latex gloves have to be taken.
- Serum and reagent spills have to be wiped off with a disinfecting solution (e.g. sodium hypochlorite 5 %) and have to be disposed of properly.
- All reagents have to be brought to room temperature (18 to 24 °C) before performing the test.
- Before pipetting all reagents should be mixed thoroughly by gentle tilting or swinging. Vigorous shaking with formation of foam should be avoided.
- It is important to pipet with constant intervals so that all the wells of the microtiter plate have the same conditions.
- When removing reagents out of the bottles care has to be taken that the stoppers are not contaminated. Further a possible mix-up has to be avoided. The content of the bottles is usually sensitive to oxidation so that they should be opened only for a short time.
- In order to avoid a carry-over or a cross-contamination separate disposable pipet tips have to be used.
- No reagents from different kit lots should be used and they should not be mixed with one another.
- All reagents have to be used within shelf life.
- In accordance with a Good Laboratory Practice (GLP) or following ISO 9001 all laboratory devices employed should be regularly checked regarding the accuracy and precision. This refers e.g. to microliter pipets and washing or reading (ELISA Reader) instrumentation.
- The contact of certain reagents especially the stopping solution and the substrate with skin, eye and mucosa has to be avoided because possible irritations and acid burns could arise and there exists a danger of intoxication.

7. Storage and Stability

Store all reagents at 4 - 8 °C.

The expiry date of each reagent is printed on the individual labels. Do not use any reagents after the expiry date has been exceeded.

The diluted washing buffer is stable for up to 4 weeks when stored at 4 - 8 °C.

8. Specimen Collection and Handling

Both serum or plasma (EDTA, heparin) can be used for the determination. Serum is separated from the blood which is aseptically drawn by venipuncture after clotting and centrifugation. The serum or plasma samples can be stored at 4 - 8 °C for up to 48 hours. They should be kept at -20°C for a longer storage. The samples should not be frozen and thawed repeatedly. Lipemic, hemolytic or bacterially contaminated samples can cause false positive or false negative results.

Patient sera must be prediluted **1:101** in **serum diluent** (e.g. 5 µL serum + 500 µL serum diluent) prior to testing.

Samples containing concentrations higher than the highest calibrator have to be diluted further with serum diluent.

In case of interference with rheumatic factors, serum preabsorption with Rf absorbant (MASTSORB Order Code: 651003) is recommended. Do **not** absorb the calibrators.

9. Assay Procedure

9.1. Preparation of Reagents

Allow all kit components and specimens to reach room temperature (RT, 18 – 24 °C) prior to use and mix well.

Washing buffer: Dissolve any crystals which may be in the bottle by warming to 37 °C and then mix well.

Dilute the concentrated **washing buffer 1:10 with distilled water** (e.g. 60 mL buffer concentrate + 540 mL distilled water). Mix thoroughly.

- Strictly follow the instructions for reliable test performance. Any changes or modifications are within the responsibility of the user.
- All reagents and samples must be brought to room temperature before use, but should not be left at this temperature for longer than necessary.
- A standard curve should be established with each assay.
- Put the unused microtiter strips back in the plastic bag and store them dry at 4 - 8 °C.

9.2. Assay Steps

Prepare a sufficient amount of microtiter wells for calibrators, controls and samples.

Note: Other incubation conditions might be possible. In case of modifications of the recommended test procedure (e.g. incubation temperature 37 °C instead of RT) the user has to validate assay performance.

1. Pipette **100 µL** each of the **diluted (1:101) samples** and the **ready to use calibrators** into the appropriate wells.
2. Cover plate with the enclosed plate sealing foil and **incubate** at room temperature for **60 minutes**.
3. Discard the contents of the microwells and wash **3 times** with **300 µL** of **diluted washing buffer**. Afterwards remove residues of the washing solution by gentle tapping of the microtiter plate on a paper towel.
4. Pipette **100 µL** of **enzyme conjugate** solution into each well.
5. Cover plate with plate sealing foil and **incubate** for **30 minutes** at room temperature.
6. Discard the contents of the microwells and wash **3 times** with **300 µL** of **diluted washing buffer**. Afterwards remove residues of the washing solution by gentle tapping of the microtiter plate on a paper towel.
7. Dispense **100 µL** of **TMB substrate** into each well.
8. Cover plate with the plate sealing foil and **incubate** for **20 minutes** in the dark (e.g. drawer) at room temperature.
9. Add **100 µL** of **stopping solution** to each well.
10. After thorough mixing and wiping the bottom of the plate, **read the optical density at 450 nm** and calculate the results. Blank against air. A bichromatic measurement using a reference wavelength of 600 - 690 nm is recommended.

The developed colour is stable for at least 60 minutes. Read optical densities during this time.

10. Results and Interpretation

Example

	OD 450 nm	corrected OD	Mean OD Value
Blank	0.015		
Calibrator 1 (negative)	0.036/0.039	0.021/0.024	0.023
Calibrator 2 (cut-off)	0.605/0.587	0.590/0.572	0.581
Calibrator 3 (weak positive)	1.177/1.210	1.162/1.195	1.179
Calibrator 4 (positive)	1.975/2.017	1.960/2.002	1.981

The table above should be considered as an example which was achieved under arbitrary temperature and environmental conditions. These data do NOT describe **reference values** which have to be found in other laboratories in the same way!

10.1. Qualitative Calculation

The calculated OD values for patient sera as mentioned above are compared with the value for the cut-off calibrator. If the value of the sample is higher, then it should be read as positive.

A value below the cut-off calibrator should be read as negative. It seems reasonable to define a range of $\pm 20\%$ around the value of the cut-off as a grey zone. It is recommended to repeat results laying within the grey zone using the same serum or a new sample of the same patient, taken after 2 - 4 weeks. Both samples should be measured in parallel in the same run.

The positive calibrator must show an absorption value at least double the value received by the cut-off calibrator.

10.2 Quantitative Calculation

The ready to use calibrators of the EBV (EBNA-1) antibody ELISA are defined and values expressed are in arbitrary units (U/mL). This gives access to an exact and reproducible quantification and in consequence patient antibody titer monitoring is possible. Concentration values for calibrators are printed on the labels of the vials.

A standard curve is plotted by entering the mean absorbance value of the calibrators on the Y-axis and the corresponding concentration on the X-axis using graph paper. The concentration of the patient samples can then be read directly from the graph.

The calculation of the result can be performed using a computer and a suitable software program.

11. Assay Performance

The assay characteristics of the MASTAZYME EBV (EBNA-1) IgG / IgM / IgA ELISAs have been established and assessed according the European IVD directive. Detailed validation data can be provided on special request.

12. References

1. Chow, K.C. et al. Serum responses to the combination of Epstein-Barr virus antigens from both latent and acute phases in nasopharyngeal carcinoma: complementary test of EBNA-1 with EA-D. *Cancer Epidemiol. Biomarkers Prev.*, **6**: 363 (1997).
2. Debyser, Z. et al. Comparative evaluation of three ELISA techniques and an indirect immunofluorescence assay for the serological diagnosis of Epstein-Barr virus infection. *Clin. Diagn. Virol.*, **8**: 71 (1997).
3. Dobec, M. et al. Evaluation of a new Epstein-Barr virus Combi Test for rapid serologic diagnosis of infectious mononucleosis. *Zentralbl. Bakteriol.*, **284**: 565 (1996).
4. Dopatka, H.D. et al. Compact Epstein-Barr virus diagnosis based on a defined antigen mix and specific IgA. *Res. Virol.*, **147**: 53 (1996).
5. Korycakova, L. Epstein-Barr virus antibodies: review of results from repeated examinations in 208 hemodialyzed patients. *Vnitr. Lek.*, **41**: 377 (1995).
6. Nebel-Schickel, H. et al. Anti-EBNA-1 (carboxy-half) IgG antibodies as a seroepidemiological marker for Epstein-Barr virus infection. *Beitr. Infusionsther. Transfusionsmed.*, **32**: 134 (1994).
7. Obel, N. et al. Serological and clinical findings in patients with serological evidence of reactivated Epstein-Barr virus infection. *APMIS*, **104**: 424 (1996).
8. Rivero, N. et al. Evaluation of the detection of IgM by EIA against the p18 protein of the IgG capsid against EBNA in the diagnosis of acute Epstein-Barr infection. *Enferm. Infec. Micro-biol. Clin.*, **16**: 45 (1998).
9. Ternyak, G. et al. The serological signs of the Epstein-Barr virus (EBV) activity in the elderly. *Acta Microbiol. Immunol. Hung.*, **44**: 133 (1997).
10. Weissbrich, B. The use of semi-automated EBV IgG avidity determination for the diagnosis of infectious mononucleosis. *J. Med. Virol.*, **54**: 145 (1998).

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12. Bibliographie

1. Chow, K.C. et al. Serum responses to the combination of Epstein-Barr virus antigens from both latent and acute phases in nasopharyngeal carcinoma: complementary test of EBNA-1 with EA-D. *Cancer Epidemiol. Biomarkers Prev.*, 6: 363 (1997).
2. Debyser, Z. et al. Comparative evaluation of three ELISA techniques and an indirect immunofluorescence assay for the serological diagnosis of Epstein-Barr virus infection. *Clin. Diagn. Virol.*, 8: 71 (1997).
3. Dobec, M. et al. Evaluation of a new Epstein-Barr virus Combi Test for rapid serologic diagnosis of infectious mononucleosis. *Zentralbl. Bakteriol.*, 284: 565 (1996).
4. Dopatka, H.D. et al. Compact Epstein-Barr virus diagnosis based on a defined antigen mix and specific IgA. *Res. Virol.*, 147: 53 (1996).
5. Korycakova, L. Epstein-Barr virus antibodies: review of results from repeated examinations in 208 hemodialyzed patients. *Vnitr. Lek.*, 41: 377 (1995).
6. Nebel-Schickel, H. et al. Anti-EBNA-1 (carboxy-half) IgG antibodies as a seroepidemiological marker for Epstein-Barr virus infection. *Beitr. Infusionsther. Transfusionsmed.*, 32: 134 (1994).
7. Obel, N. et al. Serological and clinical findings in patients with serological evidence of reactivated Epstein-Barr virus infection. *APMIS*, 104: 424 (1996).
8. Rivero, N. et al. Evaluation of the detection of IgM by EIA against the p18 protein of the IgG capsid against EBNA in the diagnosis of acute Epstein-Barr infection. *Enferm. Infec. Micro-biol. Clin.*, 16: 45 (1998).
9. Ternyak, G. et al. The serological signs of the Epstein-Barr virus (EBV) activity in the elderly. *Acta Microbiol. Immunol. Hung.*, 44: 133 (1997).
10. Weissbrich, B. The use of semi-automated EBV IgG avidity determination for the diagnosis of infectious mononucleosis. *J. Med. Virol.*, 54: 145 (1998).

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Laboratoriums-Präparate GmbH
MAST GROUP Ltd.