Double/Triquetra Nichrome Loops

CE 

Double/Triquetra Nichrome Loops

ITALIANO	3
FRANÇAIS	4
ENGLISH	5
中文.....	6
DEUTSCH	7
ESPAÑOL	8
PORTUGUÊS	9
SRPSKA	10
TÜRK	11
한국어	12
日本人	13
ČESKY	14
ROMÂNĂ	15
DANSK	16
MAGYAR	17
NORSK	18
РУССКИЙ	19
SVENSKA	20
EESTI	21
LATVIEŠU	22
LIETUVIŲ	23
SLOVENSKÉ	24
УКРАЇНСЬКА	25

Copan

Copan Wasp S.r.l.
Via Achille Grandi, 32
25125 Brescia - Italy
Tel: +39 030 2687211/218
Fax: +39 0302659932

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Double/Triquetra Nichrome Loops

FOR INOCULATION AND STREAKING WITH WASP®

ENGLISH

INTENDED PURPOSE

For automatic microbiological samples inoculation and streaking.

INTENDED USE

The **Double/Triquetra Nichrome Loops** are reusable tools intended to be used with the WASP® automation to transfer a specified amount of microbiological specimens from the specimen container to the media to be processed. The WASP® Loops have been designed to inoculate broth, to streak culture media plate and to smear microscope slides.

PRODUCT DESCRIPTION

The **Double/Triquetra Nichrome Loops** are available with different loop size, refers to the table below:

REF	Name	Description	Pack Size
WR086-03-0600	Double Nichrome Loop 1 µl rounding	Two loops with 1 µl ring moulded in a yellow plastic base	2 units for box
WR086-03-0601	Double Nichrome Loop 10 µl	Two loops with 10 µl ring moulded in a blue plastic base	2 units for box
WR086-03-0602	Double Nichrome Loop 30 µl	Two loops with 30 µl ring moulded in a red plastic base	2 units for box
WR086-03-0603	Twin Double Nichrome Loop 1 µl rounding	Four loops with 1 µl ring moulded in a yellow plastic base	1 unit for box
WR086-03-0716	Twin Triquetra Nichrome Loop 1 µl rounding	Six loops with 1 µl ring moulded in yellow plastic base	2 units for box
WR086-03-0717	Triquetra Nichrome Loop 30 µl	Three loops with 30 µl ring moulded in red plastic base	1 unit for box
WR086-03-0718	Triquetra Nichrome Loop 10 µl	Three loops with 10 µl ring moulded in blue plastic base	1 unit for box
WR086-03-0719	Triquetra Nichrome Loop 1 µl rounding	Three loops with 1 µl ring moulded in yellow plastic base	1 unit for box

PRECAUTIONS

- Do not use **Double/Triquetra Nichrome Loops** if damaged or not intact.
- For *in vitro* use only.
- Carefully follow the instructions for use. The manufacturer cannot be held responsible for any unauthorised or unqualified use of the product.
- For professional use only. The product must be used only by adequately instructed personnel.
- Loops may turn brownish when passed through the incinerator. Brownish maculas are not to be considered a product defect and do not affect in any way the product performance.
- For disposal of the **Double/Triquetra Nichrome Loops** follow the Laboratory Good Practices.

INSTRUCTIONS FOR USE

- Before uses verify the alignment of the loop with the dedicated tool. Refer to

Double Loop Adjustment Tool (WSPD086-1008) or Triquetra Loop Adjustment Tool (WSPD086-1001) packing insert.

- For instruction for use (including the wash loop procedure), please refer to WASP® - Operator Manual.

PRODUCT DETERIORATION

- COPAN guarantees a minimum mechanical life cycle of 15.000 processed plates for Double Loops and 22.500 for Triquetra Loops. This number has been estimated considering one loop sterilization for each processed plate and it may change according to the protocol settings.
- The lifespan of the loop may be affected by several variables including clinical sample consistency, culture plate and broth composition and streaking pattern.

TABLE OF SYMBOLS

Symbol	Meaning
	Manufacturer
	Batch code
	Catalogue number
	Production date
	In Vitro Diagnostic Medical Device
	Consult the operating instructions supplied with the device or available in electronic format, and which can be identified by the e-IFU indicator on the packaging label
	Quantity (expressed in terms of numerical count)
Rx Only	For US Only Caution: US Federal Law restricts this device to sale by or on the order of a licensed practitioner

In case of serious incident, contact the manufacturer and the Competent Authority.

Copan Wasp S.r.l.
Via Achille Grandi, 32
25125 Brescia - Italy
Tel: +39 030 2687211/218
Fax: +39 0302659932

Email: wasp@copangroup.com
Website: www.copangroup.com

North American Distributor:
Copan Diagnostics Inc.
Murrieta, CA 92562 USA
Tel: 8002164016

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