Vitassay qPCR

Dengue

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- LightCycler Z480, 480II (Roche)
- Cobas Z480 (Roche)
- 7500 Fast Real-Time PCR System (Applied Biosystems) ^{II}
- StepOne™ Real-Time PCR System (Applied Biosystems) ^{II}
- CFX96™ Real-Time PCR Detection System (Bio-Rad)
- AriaMx Real-Time PCR System (Agilent Technologies)
- DTPrime Real Time Detection Thermal Cycler (DNA-Technology)
- DTLite Real Time PCR System (DNA-Technology)
- Rotor-Gene® Q (Qiagen)^I
- SmartCycler® (Cepheid)^I

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Termocicladores con bloque de bajo perfil	Termocicladores con bloque de alto perfil
Agilent Technologies	Abbott
AriaMx Real-Time PCR System	Abbott m2000 RealTime System
Applied Biosystems	Applied Biosystems
7500 Fast Real-Time PCR System	7300 Real-Time PCR System
7500 Fast Dx Real-Time PCR System	7500 Real-Time PCR System
QuantStudio™ 12K Flex 96-well Fast	7900 HT Real-Time PCR System
QuantStudio™ 6 Flex 96-well Fast	ABI PRISM 7000
QuantStudio™ 7 Flex 96-well Fast	ABI PRISM 7700
QuantStudio™ 5 Real-Time PCR System	QuantStudio™ 12K Flex 96-well
QuantStudio™ 3 Real-Time PCR System	QuantStudio™ 6 Flex 96-well
StepOne Plus™ Real-Time PCR System	QuantStudio™ 7 Flex 96-well
StepOne™ Real-Time PCR System	QuantStudio™ 5 Real-Time PCR
ViiA™ 7 Fast Real-Time PCR System	QuantStudio™ 3 Real-Time PCR System
BIONEER	ViiA™ 7 Real-Time PCR
Exicycler™ 96	Analytik Jena Biometra
Bio-Rad	TOptical
CFX96™ Real-Time PCR Detection System	qTOWER 2.0
Mini Opticon™ Real-Time PCR Detection System	BIONEER
Cepheid	Exicycler™ 96
SmartCycler®	Bio-Rad
Qiagen	CFX96™ Deep Well Real-Time PCR Detection
Rotor-Gene® Q	iCycler iQ™ Real-Time PCR
Roche	iCycler iQ™ 5 Real-Time PCR
LightCycler®480 Real-Time PCR System	MyIQ™ Real-Time PCR Detection System
LightCycler®96 Real-Time PCR System	MyIQ™ 2Real-Time PCR Detection System
Cobas z480 Analyzer	Cepheid
	SmartCycler®
	DNA-Technology
	DTlite Real-Time PCR System*
	DTprime Real-time Detection Thermal Cycler*
	Eppendorf
	Mastercycler™ ep realplex
	Qiagen
	Rotor-Gene® Q
	Stratagene / Agilent Technologies
	Mx3000P™ Real Time PCR System
	Mx3005P™ Real Time PCR System

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Intended use

Vitassay qPCR Dengue allows the qualitative detection of Dengue virus by real-time RT-PCR in clinical samples. The product is intended for use in the diagnosis of Dengue virus infections alongside clinical data of the patient and other laboratory tests outcomes.

References

Vitassay qPCR Dengue 4 x 8-well strip, low profile	7041002
Vitassay qPCR Dengue 4 x 8-well strip, high profile	7042002

Materials/reagents provided

Reference	Reagent/Material	Colour	Amount
7041S002/ 7042S002	Dengue Virus strips high/low profile	-	4x8-well strip
7C002	Dengue Virus Positive Control	red	1 vial
7001A	PCR grade water	white	1 vial x 1 mL
7002B	Rehydratation Buffer	green	1 vial x 1,8 mL
7003N	Negative control	yellow	1 vial x 1 mL
7004O	Optical caps	-	4x8 cap strip

Transport and storage

- The reagents and the test can be shipped and stored at 2-40°C until expiration date stated in the label.
- The resuspended positive control should be stored at -20°C. In order to avoid repeated freeze/thaw cycles, it is recommended to distribute the content in different aliquots.
- Keep all reagents in the darkness.

Additional equipment and material required

- Real-time PCR instrument (thermocycler) (Attached I)
- RNA extraction kit
- Centrifuge for 1.5 mL tubes
- Vortexer
- Micropipettes (1-20 µL, 20-200 µL)
- Filter tips
- Powder-free disposal gloves

Summary

Dengue is a mosquito-borne viral disease that has rapidly spread in recent years. Dengue virus is transmitted by female mosquitoes mainly of the species *Aedes aegypti* and, to a lesser extent, *Ae. albopictus*. One study, of the prevalence of dengue, estimates that 3.9 billion people, in 128 countries, are at risk of infection with dengue viruses.

Many people may experience no symptoms during a mild case of dengue fever. When symptoms do occur, they usually begin four to 10 days after you are bitten by an infected mosquito. Signs and symptoms of dengue fever most commonly include: high fever, headaches, muscle, bone and joint pain, pain behind your eyes. You might also experience widespread rash, nausea and vomiting and rarely, minor bleeding from your gums or nose. In some cases, symptoms worsen and can become life-threatening. Blood vessels often become damaged and leaky. And the number of clot-forming cells (platelets) in your bloodstream drops.

Dengue can be diagnosed by isolation of the virus, by serological tests (antigen and antibodies detection), or by molecular methods. Diagnosis of acute or recent dengue infection can be established by testing serum samples during the first 5 days of symptoms and/or early convalescent phase. Methods such as one-step real time RT-PCR are now widely used to detect dengue viral genes in acute-phase serum samples. This detection coincides with the viremia and the febrile phase of illness onset. It's important to consider that serological methods have cross reactivity with other flaviviruses including West Nile virus (WNV), St. Louis encephalitis virus (SLE), Japanese encephalitis virus (JEV) and yellow fever virus (YFV).

Principle of the test

Vitassay qPCR Dengue is based on the real-time amplification of the 3' non-coding region encoded by the Dengue virus genome. The viral RNA extracted is transcribed into cDNA using a specific primer by reverse transcription step followed immediately in the same well by polymerase chain reaction. The presence of Dengue virus is detected by an increase in observed fluorescence during the reaction upon hydrolysis of the fluorescent probe.

Vitassay qPCR Dengue is a ready-to used test which contains in each well all the necessary reagents for real-time PCR assay in a stabilized format. In addition, an internal control allows the detection of a possible reaction inhibition. The amplification of the target sequence is detected through the FAM channel whereas the internal control (IC) in HEX, VIC or JOE channel (depending on the equipment used).

Precautions

- For *in vitro* diagnostic use.
- Do not use after expiration date.
- Do not mix components from different kits and/or lots.
- Do not use if package is open or damaged.
- Do not use the kit if the desiccant from the different reagents is not present or broken or if the foil has been broken or damaged.
- Protect the kit against humidity. Prolonged exposure to humidity may affect product performance.
- Follow Good Laboratory Practices. Wear protective clothing, use disposal gloves, goggles and mask.
- Do not eat, drink or smoke in the working area.
- The laboratory process must be one-directional, it should begin in the Extraction Area and then move to the Amplification and Detection Areas. Do not return samples, equipment and reagents to the area in which the previous step was performed.
- Specimens and reagents/materials that have been exposed to them must be treated as potentially infectious. Take necessary precautions during the collection, storage, treatment and disposal of samples.
- Regular decontamination of commonly used equipment is recommended, especially micropipettes and work surfaces.

Procedures

Specimen collection, processing and RNA extraction

For pretreatment and nucleic acid isolation, it is recommended to use your optimized manual or automatic system compatible with real-time PCR technology. The assay has been validated with the following extraction kits:

Maxwell® 16 Viral Total Nucleic Acid Purification Kit (Promega).

QIAamp Viral RNA Mini Kit, QIAcube instrument (Qiagen).

Invisorb® Spin Universal Kit (Stratec).

Positive control preparation

Reconstitute the lyophilized Dengue Virus Positive Control (red tube) with the 100 µL of PCR grade water (white tube). To ensure a complete resuspension, vortex the tube thoroughly. After first use, dispense into aliquots in order to avoid multiple freeze-thaw cycles, and store them at -20°C.

This component contains high copies number template and is a very significant contamination risk. Therefore, we recommend open and manipulate it in a separate laboratory area away from the other components and samples.

Reaction setup

- Separate the number of required reactions including samples and controls. Remember that one positive and one negative control must be included in each run.
- Peel off protective aluminium seal from the strips.
- Pipette 15 µL of Resuspension buffer (green tube) and add them into each well.
- Pipette 5 µL of RNA sample, negative and positive controls and add them into each well.
- Cover the wells with the caps provided. Spin down briefly (optional).
- Place the strips in the Real-time PCR instrument.

Programme your thermocycler

Set your thermocycler following the conditions below:

Step	Temperature	Time	Cycles
Reverse transcription	45°C	15 min	1
Initial denaturation	95°C	2 min	1
Denaturation	95°C	10 sec	
Annealing/Extension (Data collection*)	60°C	50 sec	45

Set the fluorescence data collection during the extension step (*) through the FAM (Dengue virus) and HEX, JOE or VIC channels (Internal Control (IC)). If you use the Applied Biosystems 7500 Fast, the Applied Biosystems StepOne™ or the Stratagene Mx3005P™ check that passive reference option ROX is none (attached II).

Analysis and interpretation of results

The analysis of the results is done by the software itself of the used real-time PCR system following manufacturer's instructions.

For a valid diagnostic test run, the following control conditions must be met:

Positive control

The positive controls used in each run, must show an amplification curve for Dengue virus, which validates the reaction.

Negative control

The negative controls included in each run, must show the absence of signal for Dengue virus, which validates the reaction.

The experiment seems to be fail if there is signal of amplification in negative control or absence of signal in the positive well. The assay should be repeated.

The result interpretation is summarized in the following table:

Dengue Virus FAM	Internal Control HEX	Negative Control	Positive Control	Interpretation
+	+/-	-	+	Virus Dengue Positive
-	+	-	+	Virus Dengue Negative
+	+	+	+	Invalid
-	-	-	-	Invalid

(+) Positive: Amplification signal

(-) Negative: No amplification signal

If the negative samples not show a positive result for the internal control, they should be retested from the diluted original sample 1:10 or the nucleic acid extraction has to be repeated due to possible problems caused by PCR inhibitors.

Quality Control

In order to confirm the appropriate performance of the molecular diagnostic technique, an Internal Control (IC) is included in each reaction. Besides, a positive and a negative control must be included in each assay to interpret the results correctly.

Performance evaluation

Clinical sensitivity and specificity

Tetassay qPCR Dengue was evaluated with the programs: QCMD 2017 Dengue Virus EQA programme (DENVRNA17) and QCMD 2018 Dengue Virus EQA programme (DENVRNA18). Both programmes consisted of panels of 10 samples in transport

medium positive or not to the studied pathogen. Vitassay qPCR Dengue correctly detected all samples from both panels.

These results indicate that Vitassay qPCR Dengue shows a high sensitivity and specificity to detect the Dengue virus.

Analytical sensitivity

The analytical sensitivity was determined by analysis of 10-fold dilution series of Dengue virus template ranging from 10^7 to 10^1 copies/rxn. This assay has a detection limit of ≥ 10 viral RNA copies per reaction.

Analytical specificity

The analytical specificity for Dengue virus was tested within the panel of following microorganisms, where no cross-reactivity was seen between any of the species:

Dengue Virus		
Zika Virus strain 11474/16 (French Polynesia)	Plasmodium falciparum 3D7	West Nile virus Heja
Zika Virus strain 11468/16(French Polynesia)	Japanese encephalitis	West Nile virus Ug37
Zika Virus (African)	St Louis Encephalitis virus	Yellow Fever virus strain 17D
Zika virus strain PF13/251013- 18 (Asian)	West Nile virus strain NY99	Trypanosoma cruzi
Chikungunya virus S27 Petersfield		

Analytical reactivity

The reactivity of Vitassay qPCR Dengue was confirmed by the real-time amplification using Dengue 1 virus strain Hawaii, Dengue 2 virus strain New Guinea C, Dengue 3 virus strain H87 and Dengue 4 virus strain H241 as template.

Compatibles real-time PCR equipment

Vitassay qPCR Dengue has been validated on the following equipments:

- LightCycler Z480, 480II (Roche)
- Cobas Z480 (Roche)
- 7500 Fast Real-Time PCR System (Applied Biosystems) ^{II}
- StepOne™ Real-Time PCR System (Applied Biosystems) ^{II}
- CFX96 TouchTM Real-Time PCR Detection System (Bio-Rad)
- AriaMx Real-Time PCR System (Agilent Technologies)

- DTPrime Real Time Detection Thermal Cycler (DNA-Technology)
- DTLite Real Time PCR System (DNA-Technology)
- Rotor-Gene® Q (Qiagen)¹
- SmartCycler® (Cepheid)¹

I: For Rotor-Gene® Q and SmartCycler® thermocyclers the product should be reconstituted following the procedure and transferred into specific Rotor-Gene® Q and/or SmartCycler tubes.

II: When using the Applied Biosystems 7500 Fast with strips it is recommend to place a plate holder for the tubes (Ref. PN 4388506).

Limitations

- This test provides a presumptive diagnosis of Dengue virus infection. All results must be interpreted together with other clinical information and laboratory findings available to the physician.
- This assay was tried with samples in transport medium. The use of other samples has not been established.
- The quality of the test depends on the quality of the sample; proper RNA from clinical specimens must be extracted. Unsuitable collection, storage and/or transport of specimens may give false negative results.
- Extremely low levels of target below the limit of detection may be detected, but results may not be reproducible.
- There is a possibility of false positive results due to cross-contamination by Dengue virus, either samples containing high concentrations of target RNA or contamination due to PCR products from previous reactions.

Attached I: Compatibility of the most common real-time PCR equipment

The most common thermocyclers are listed in the following table separated by tube type. If you do not find your thermocycler, please contact with your supplier.

Low profile Block Thermocyclers	High profile Block Thermocyclers
Agilent Technologies	Abbott
AriaMx Real-Time PCR System	Abbott m2000 RealTime System
Applied Biosystems	Applied Biosystems
7500 Fast Real-Time PCR System	7300 Real-Time PCR System
7500 Fast Dx Real-Time PCR System	7500 Real-Time PCR System
QuantStudio™ 12K Flex 96-well Fast	7900 HT Real-Time PCR System
QuantStudio™ 6 Flex 96-well Fast	ABI PRISM 7000
QuantStudio™ 7 Flex 96-well Fast	ABI PRISM 7700
QuantStudio™ 5 Real-Time PCR System	QuantStudio™ 12K Flex 96-well
QuantStudio™ 3 Real-Time PCR System	QuantStudio™ 6 Flex 96-well
StepOne Plus™ Real-Time PCR System	QuantStudio™ 7 Flex 96-well
StepOne™ Real-Time PCR System	QuantStudio™ 5 Real-Time PCR
ViiA™ 7 Fast Real-Time PCR System	QuantStudio™ 3 Real-Time PCR System
BIONEER	ViiA™ 7 Real-Time PCR
Exicycler™ 96	Analytik Jena Biometra
Bio-Rad	TOptical
CFX96™ Real-Time PCR Detection System	qTOWER 2.0
Mini Opticon™ Real-Time PCR Detection System	BIONEER
Cepheid	Exicycler™ 96
SmartCycler®	Bio-Rad
Qiagen	CFX96™ Deep Well Real-Time PCR Detection
Rotor-Gene® Q	iCycler iQ™ Real-Time PCR
Roche	iCycler iQ™ 5 Real-Time PCR
LightCycler ®480 Real-Time PCR System	MyIQ™ Real-Time PCR Detection System
LightCycler ®96 Real-Time PCR System	MyIQ™ 2Real-Time PCR Detection System
Cobas z480 Analyzer	Cepheid
	SmartCycler®
	DNA-Technology
	DTlite Real-Time PCR System
	DTprime Real-time Detection Thermal Cycler
	Eppendorf
	Mastercycler™ ep realplex
	Qiagen
	Rotor-Gene® Q
	Stratagene / Agilent Technologies
	Mx3000P™ Real Time PCR System
	Mx3005P™ Real Time PCR System

* See Attached III to configure exposure settings.

Attached II: Detection channels of most common real time PCR equipment

The fluorescence detection channels of some of most common Real Time PCR Thermocyclers are specified in the following table:

REAL-TIME PCR THERMOCYCLER	Vitassay CHANNEL	DETECTION CHANNEL	OBSERVATIONS
Bio-Rad CFX96™	FAM	FAM	
	HEX	HEX	
	ROX	ROX	
	Cy5	Cy5	
ABI 7500 Applied Biosystems	FAM	FAM	Passive reference option ROX is none
	HEX	VIC	
	ROX	ROX	
	Cy5	Cy5	
Roche Lightcycler®480II	FAM	465/510	Colour Compensation required
	HEX	533/580	
	ROX	533/610	
	Cy5	618/660	
Smartcycler® Cepheid	FAM	Channel 1	
	HEX	Channel 2	
	ROX	Channel 3	
	Cy5	Channel 4	
Abbott m2000rt	FAM	FAM	
	HEX	VIC	
	ROX	ROX	
	Cy5	Cy5	
Mx3000P™ Mx 3005P™ Stratagene	FAM	FAM	Passive reference option ROX is none
	HEX	VIC	
	ROX	ROX	
	Cy5	Cy5	
AriaMx Agilent	FAM	FAM	
	HEX	HEX	
	ROX	ROX	
	Cy5	Cy5	
Rotor-Gene®Q Qiagen	FAM	Green	In the Channel Setup, click on the "Gain Optimisation" button and then go to "Optimise Acquiring". The fluorescence Target Sample Range has to be between 5 and 10 FI for each channel. Also select the option "Perform Optimisation Before 1st Acquisition".
	HEX	Yellow	
	ROX	Orange	
	Cy5	Red	
Exicycler™ 96 BIONEER	FAM	FAM	
	HEX	JOE	
	ROX	ROX	
	Cy5	Cy5	

Attached III: Optical measurement exposure setting

The exposure values of some thermocyclers must be adjusted for proper operation. Set exposure values as follows:

Thermocycler	Vitassay channel	Exposure values
DTlite Real-Time PCR System (DNA-Technology)	FAM	500
	HEX	500
	ROX	500
	Cy5	500
DTprime Real-time Detection Thermal Cycler (DNA-Technology)	FAM	500
	HEX	1000
	ROX	1000
	Cy5	1000

Bibliography/Bibliografía

1. Genetic and serologic properties of Dengue virus associated with an epidemic, Yap State, Micronesia, 2007.Lanciotti RS, Kosoy OL, Laven JJ, Velez JO, Lambert AJ, Johnson AJ, Stanfield SM, Duffy MR. Emerg Infect Dis. 2008 Aug;14(8):1232-9.
2. Quantitative real-time PCR detection of Dengue virus and evaluation with field-caught mosquitoes.Faye O, Faye O, Diallo D, Diallo M, Weidmann M, Sall AA. Virol J. 2013 Oct 22;10:311.
3. Detection of Dengue virus in urine.Gourinat. AC, O'Connor O, Calvez E, Goarant C, Dupont-Rouzeyrol M. Emerg Infect Dis. 2015 Jan;21(1):84-6.
4. Centers for Disease Control and Prevention. Dengue Virus (<http://www.cdc.gov/Dengue/index.html>).
5. World Health Organization. Dengue virus and potential complications (<http://www.who.int/ emergencies/Dengue-virus/en/>).

Trademarks

CFX™ and IQ5™ are registered trademarks of Bio-Rad Laboratories.

ABI®, QuantStudio™, StepOnePlus™ and ViiA™ are registered trademarks of Thermo Fisher Scientific Inc.

LightCycler® is a registered trademark of Roche.

Mx3000P™ and Mx3005™ are registered trademarks of Agilent Technologies.

Mastercycler™ is a registered trademark of Eppendorf.

Rotor-Gene® Q is a registered trademark of Qiagen.

SmartCycler® is a registered trademark of Cepheid

Symbols for IVD components and reagents/ Símbolos utilizados para componentes y reactivos IVD

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